**Aquasonic® 100 Ultrasound Transmission Gel**  
**Aquasonic CLEAR® Ultrasound Gel**  
**SCAN® Ultrasound Gel**

**REF** 01-02, 01-08, 01-34, 01-50

**REF** 03-02, 03-08, 03-34, 03-50, 03-54

**REF** 11-08, 11-28, 11-28s

- EN** Instructions for use
- ES** Instrucciones de uso
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- PT** Instruções de utilização
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- EL** Οδηγίες Χρήσης
- HU** Használati útmutatás
- PO** Instrukcja stosowania
- HE** הוראות שימוש
- JA** 取扱説明書
- KO** 사용법
- ZH** 使用说明
- AR** إرشادات الاستعمال

 Parker Laboratories, Inc.  
286 Eldridge Road  
Fairfield, NJ 07004 USA  
(973) 276-9500  
www.parkerlabs.com

EC	REP
----	-----

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**Instructions for Use:**

**Description**

Aquasonic 100 Ultrasound Transmission Gel, Aquasonic Clear Ultrasound Gel and SCAN Ultrasound Gel (Parker Laboratories Ultrasound Gel) are non-sterile, water-soluble and non-staining gels intended for external use only and are used with all commercially available ultrasound devices.

**Indications for Use**

Parker Laboratories Ultrasound Gel can be used on intact, unbroken skin and on all patients in facilities where cross contamination is of minimal concern. Discard empty dispenser after use.

Refill containers and refillable dispensers (product numbers 01-50, 03-50, 03-54, 11-28 and 11-28s) are for use on low-risk patients and procedures in non-acute care facilities. Follow the enclosed directions for use or your facility's protocol, if it is different.

**Contraindications**

Parker Laboratories Ultrasound Gel, in multi-use packaging, is for external use only and should not be used in:

- Any invasive procedures in which a device is passed through the tissue, e.g. needle aspiration, needle localization, tissue biopsy, or other surgical procedures.
- Any procedure on or near broken skin, such as fresh surgical site or an open wound.
- Patients with an immunodeficiency or on immunosuppressive therapy.
- Any neonates or critically ill pediatric patients.
- Mucous membranes.
- Critically ill patients or patients in contact-, airborne- or droplet-transmission-based precautions.

**Storage Conditions**

Store in a cool dry place. Store away from direct sunlight.

**Warnings and Precautions**

Discard product if package is damaged or evidence of contamination is present. Do not use on patients with a known sensitivity to parabens.

To avoid contamination, tips of containers should not come into contact with patient, staff or instruments.

Apply Parker Laboratories Ultrasound Gel to transducer surface, and not directly on to the patient.

**Directions for Refilling Dispenser Bottles**

Consult your facility policy before refilling from bulk container. For refilling dispenser bottles, wipe the bottleneck and dispenser tip with isopropyl alcohol or a hospital-grade disinfectant, such as Protex™ Disinfectant Wipes, just prior to filling.

Reusable containers should be washed in hot soapy water, rinsed and thoroughly dried before being refilled.

Isopropyl alcohol or a hospital-grade disinfectant, such as Protex Disinfectant Wipes, can be used to clean dispenser bottles.

Only refill product into corresponding labeled dispenser, and mark it with lot number.

Reusable bottles should be marked with the date filled and then discarded after one month.

Cracked or damaged bottles should be discarded immediately.

Reusable bottles should **not** be topped off. Contents should be consumed, or remaining gel should be disposed prior to refilling.

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© 2014 Parker Laboratories, Inc.